

## **Test Definition: HCSRN**

Hepatitis C Virus (HCV) Antibody Screen with Reflex to HCV RNA, PCR, Asymptomatic, Serum

Reporting Title: HCV Ab Scrn w/Reflex to HCV PCR, S

Performing Location: Rochester

## **Ordering Guidance:**

This test is **not intended** for testing **symptomatic** individuals (ie, diagnostic purposes). For testing such patients with or without risk factors for hepatitis C virus (HCV) infection, order HCVDX / Hepatitis C Virus (HCV) Antibody with Reflex to HCV RNA, PCR, Symptomatic, Serum.

For testing autopsy/cadaver or hemolyzed specimens, order HCCAD / Hepatitis C Virus Antibody Screen for Cadaveric or Hemolyzed Specimens, Asymptomatic, Serum for asymptomatic individuals or HCCDD / Hepatitis C Virus Antibody in Cadaveric or Hemolyzed Specimens, Symptomatic, Serum for symptomatic individuals.

For patients with acute or recent HCV infections (<3 months from time of exposure) or are repeatedly reactive by screening tests and should be confirmed by a more HCV-specific test, order HCVQN / Hepatitis C Virus (HCV) RNA Detection and Quantification by Real-Time Reverse Transcription-PCR (RT-PCR), Serum.

## **Shipping Instructions:**

If shipment will be delayed for more than 24 hours, freeze serum at -70 degrees C until shipment on and transport on dry ice.

#### **Necessary Information:**

Date of collection is required.

## **Specimen Requirements:**

**Patient Preparation:** For 24 hours before specimen collection, patient should **not** take multivitamins or dietary supplements (eg, hair, skin, and nail supplements) containing biotin (vitamin B7).

Supplies: Sarstedt Aliquot Tube, 5 mL (T914)

Collection Container/Tube: Serum gel
Submission Container/Tube: Plastic vial

**Specimen Volume:** 1.3 mL **Collection Instructions:** 

- 1. Centrifuge blood collection tube per manufacturer's instructions (eg, centrifuge and aliquot within 2 hours of collection for BD Vacutainer tubes).
- 2. Aliquot serum into plastic vial.

#### Forms:

If not ordering electronically, complete, print, and send 1 of the following:

- -Gastroenterology and Hepatology Test Request (T728)
- -Infectious Disease Serology Test Request (T916)

Specimen Type	Temperature	Time	Special Container
Serum SST	Frozen (preferred)	84 days	
	Refrigerated	6 days	



# **Test Definition: HCSRN**

Hepatitis C Virus (HCV) Antibody Screen with Reflex to HCV RNA, PCR, Asymptomatic, Serum

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
HCVA5	HCV Ab Screen, S	Alphanumeric		40726-2

LOINC® and CPT codes are provided by the performing laboratory.

## **Supplemental Report:**

No

## **CPT Code Information:**

86803

G0472 (if appropriate for government payers)

87522 Hepatitis C, quantification (if appropriate)

## **Reflex Tests:**

Test Id	Reporting Name	CPT Units	CPT Code	Always Performed	Available Separately
HCVQN	HCV RNA Detect/Quant, S	1	87522	No	Yes

## **Result Codes for Reflex Tests:**

Test ID	Result ID	Reporting Name	Туре	Unit	LOINC®
HCVQN	97291	HCV RNA Detect/Quant, S	Alphanumeric	IU/mL	11011-4

## **Reference Values:**

Negative

See Viral Hepatitis Serologic Profiles