

Reporting Title: Acetaminophen, S
Performing Location: Rochester

Specimen Requirements:

Collection Container/Tube:

Preferred: Serum gel

Acceptable: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 0.5 mL

Collection Instructions:

- Serum gel tubes should be centrifuged within 2 hours of collection.
- Red-top tubes should be centrifuged, and the serum aliquoted into a plastic vial within 2 hours of collection.

Forms:

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	7 days	
	Frozen	180 days	
	Ambient	24 hours	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
ACMA	Acetaminophen, S	Numeric	mcg/mL	3298-7

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

80143

Reference Values:

Therapeutic: 10-30 mcg/mL

Toxic: >150 mcg/mL 4 hours after dose