

Reporting Title: Tobramycin, Trough, S  
Performing Location: Rochester

Specimen Requirements:

Collection Container/Tube:

Preferred: Serum gel

Acceptable: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 0.5 mL

Collection Instructions:

- 1. Draw blood no more than 30 minutes before next scheduled dose.
- 2. Serum gel tubes should be centrifuged within 2 hours of collection.
- 3. Red-top tubes should be centrifuged, and the serum aliquoted into a plastic vial within 2 hours of collection.

Forms:

If not ordering electronically, complete, print, and send a [Therapeutics Test Request](#) (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	7 days	
	Frozen	28 days	
	Ambient	72 hours	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
TOBTA	Tobramycin, Trough, S	Numeric	mcg/mL	4059-2

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

80200

Reference Values:

Therapeutic: <2.0 mcg/mL

Toxic: >2.0 mcg/mL