

Reporting Title: Tobramycin, Random, S
Performing Location: Rochester

Ordering Guidance:
Serum for a peak level should be drawn 30 to 60 minutes after last dose (order TOBPA / Tobramycin, Peak, Serum).

Serum for a trough level should be drawn no more than 30 minutes before next scheduled dose (order TOBTA / Tobramycin, Trough, Serum).

Specimen Requirements:

Preferred: Serum gel
Acceptable: Red top
Specimen Volume: 0.5 mL

- Collection Instructions:**
- 1. Serum gel tubes should be centrifuged within 2 hours of collection.
 - 2. Red-top tubes should be centrifuged and aliquoted within 2 hours of collection.

Forms:
If not ordering electronically, complete, print, and send a [Therapeutics Test Request](#) (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	7 days	
	Frozen	28 days	
	Ambient	72 hours	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
TOBRA	Tobramycin, Random, S	Numeric	mcg/mL	35670-9

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:
No

CPT Code Information:
80200

Reference Values:
TOBRAMYCIN, PEAK
Therapeutic: 3.0-12.0 mcg/mL
Toxic: >12.0 mcg/mL

TOBRAMYCIN, TROUGH

Therapeutic: <2.0 mcg/mL

Toxic: >2.0 mcg/mL