

**Reporting Title:** BCR/ABL1, p210, Quant, Reflex  
**Performing Location:** Rochester

**Specimen Requirements:**

Only orderable as a reflex. For more information see BCRFX /BCR/ABL1 Qualitative Diagnostic Assay with Reflex to BCR/ABL1 p190 Quantitative Assay or BCR/ABL1 p210 Quantitative Assay, Varies.

**Submit only 1 of the following specimens:**

**Specimen Type:** Whole blood

**Container/Tube:**

**Preferred:** Lavender top (EDTA)

**Acceptable:** Yellow top (ACD)

**Specimen Volume:** 10 mL

**Collection Instructions:**

- 1. Invert several times to mix blood.
- 2. Send whole blood specimen in original tube. **Do not aliquot.**
- 3. Label specimen as blood.

**Specimen Type:** Bone marrow

**Container/Tube:**

**Preferred:** Lavender top (EDTA)

**Acceptable:** Yellow top (ACD)

**Specimen Volume:** 4 mL

**Collection Instructions:**

- 1. Invert several times to mix bone marrow.
- 2. Send bone marrow specimen in original tube. **Do not aliquot.**
- 3. Label specimen as bone marrow.

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	72 hours	PURPLE OR PINK TOP/EDTA
	Ambient	72 hours	PURPLE OR PINK TOP/EDTA

**Supplemental Report:**

No

**CPT Code Information:**

81206

**Reference Values:**

Only orderable as a reflex. For more information see BCRFX / BCR/ABL1 Qualitative Diagnostic Assay with Reflex to BCR/ABL1 p190 Quantitative Assay or BCR/ABL1 p210 Quantitative Assay, Varies.

**Test Definition: B210R**

BCR/ABL1, p210, mRNA Detection, Reverse  
Transcription-PCR (RT-PCR), Quantitative,  
Reflex, Varies

---