

**Reporting Title:** Bean Kidney IgG  
**Performing Location:** Eurofins Viracor

**Specimen Requirements:**

**Specimen Type:** Serum

**Container/Tube:** Red Top

**Specimen Volume:** 0.5 mL

**Collection Instructions:** Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Frozen	365 days	
	Ambient	7 days	

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
FBKG	Bean Kidney IgG	Alphanumeric		60354-8

LOINC® and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

**CPT Code Information:**

86001

**Reference Values:**

Reference ranges have not been established for food-specific IgG tests.