

# **Test Definition: FFVOX**

Fluvoxamine (Luvox)

Reporting Title: Fluvoxamine (Luvox)
Performing Location: Medtox Laboratories, Inc.

## **Specimen Requirements:**

Submit only 1 of the following specimens:

#### Plasma

Draw blood in a green-top sodium heparin tube(s), plasma gel tube is not acceptable. Spin down and send 3mL of plasma refrigerated in a plastic vial.

#### Serum

Draw blood in a plain, red-top tube(s), serum gel tube is not acceptable. Spin down and send 3 mL of serum refrigerated in a plastic vial.

Specimen Type	Temperature	Time	Special Container	
Varies	Refrigerated (preferred)	7 days		
	Frozen	180 days		
	Ambient	72 hours		

### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
FFVOX	Fluvoxamine (Luvox)	Alphanumeric		10988-4

LOINC® and CPT codes are provided by the performing laboratory.

# **Supplemental Report:**

No

#### **CPT Code Information:**

80332

### **Reference Values:**

Units: ng/mL

Expected fluvoxamine concentrations on recommended daily dosage regimens: 50-900 ng/mL