

**Reporting Title:** Mirtazapine (Remeron), Serum **Performing Location:** Medtox Laboratories, Inc.

## **Specimen Requirements:**

### Submit only 1 of the following specimens:

#### Plasma

Draw blood in a green-top (sodium heparin) tube(s), **plasma gel tube is not acceptable.** Spin down and send 2 mL sodium heparin plasma refrigerated in a plastic vial.

#### Serum

Draw blood in a plain red-top tube(s), **serum gel tube is not acceptable.** Spin down and send 2 mL of serum refrigerated in a plastic vial.

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	14 days	
	Frozen	180 days	
	Ambient	72 hours	

### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
Z3323	Mirtazapine	Alphanumeric		17283-3

LOINC® and CPT codes are provided by the performing laboratory.

### Supplemental Report:

No

# **CPT Code Information:**

80335

# **Reference Values:**

Reference range: 4.0 – 40.0 ng/mL

Expected steady state trough mirtazapine concentrations in patients receiving recommended daily dosages: 4.0 – 40.0 ng/mL

Toxic range not established.