

Reporting Title: Antidepressant Drug Screen

Performing Location: Medtox Laboratories, Inc.

Specimen Requirements:

Submit only 1 of the following:

- Plasma**
- Draw blood in a green-top (sodium heparin) tube(s), **plasma gel tube is not acceptable**. Spin down and send 3 mL of plasma refrigerated in a plastic vial.
- Serum**
- Draw blood in a plain red-top tube(s), **serum gel tube is not acceptable**. Spin down and send 3 mL of serum refrigerated in a plastic vial,
- Urine**
- Collect 3 mL random urine and send refrigerated in a preservative free plastic urine container.

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	7 days	
	Frozen	180 days	
	Ambient	72 hours	

Ask at Order Entry (AOE) Questions:

Test ID	Question ID	Description	Type	Reportable
FADDS	Z3422	Specimen Type	Plain Text	Yes

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
Z3422	Specimen Type	Alphanumeric		31208-2
Z3423	Antidepressant Screen	Alphanumeric		Not Provided
Z3424	Amitriptyline	Alphanumeric		Not Provided
Z3425	Nortriptyline	Alphanumeric		Not Provided
Z3426	Imipramine	Alphanumeric		Not Provided
Z3427	Desipramine	Alphanumeric		Not Provided
Z3428	Sertraline	Alphanumeric		Not Provided
Z3429	Desmethylsertraline	Alphanumeric		Not Provided
Z3430	Fluoxetine	Alphanumeric		Not Provided
Z3431	Norfluoxetine	Alphanumeric		Not Provided
Z3432	Clomipramine	Alphanumeric		Not Provided
Z3433	Desmethylclomipramine	Alphanumeric		Not Provided
Z3434	Doxepin	Alphanumeric		Not Provided

Z3435	Desmethyldoxepin	Alphanumeric		Not Provided
Z3436	Maprotiline	Alphanumeric		Not Provided
Z3438	Trimipramine	Alphanumeric		Not Provided
Z3439	Paroxetine	Alphanumeric		Not Provided
Z3440	Cyclobenzaprine	Alphanumeric		Not Provided
Z3437	Protriptyline	Alphanumeric		Not Provided

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

- 80333
- 80337
- 80369

Reference Values:

Antidepressant screen includes the analysis for:
Amytriptyline, Clomipramine and Desmethyldomipramine, Cyclobenzaprine, Desipramine, Doxepin and
Desmethyldoxepin, Fluoxetine and Norfluoxetine, Imipramine, Maprotiline, Nortriptyline, Paroxetine, Protriptyline,
Sertraline and Desmethylertraline, Trimipramine.