

Reporting Title: Cryptococcus Ag Screen, LFA, U
Performing Location: Rochester

Specimen Requirements:

- Supplies:** Urine Tubes, 10 mL (T068)
Submission Container/Tube: Plastic, 10-mL urine tube
Specimen Volume: 1 mL
Collection Instructions:
1. Collect a random urine specimen.
2. No preservative.

Forms:

If not ordering electronically, complete, print, and send [Infectious Disease Serology Test Request](#) (T916) with the specimen.

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	14 days	
	Frozen	14 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
604095	Cryptococcus Ag Screen, LFA, U	Alphanumeric		16693-4

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

- 87899
87899 (if appropriate)

Reflex Tests:

Test Id	Reporting Name	CPT Units	CPT Code	Always Performed	Available Separately
ULFAT	Cryptococcus Ag Titer, LFA, U	1	87899	No	No

Result Codes for Reflex Tests:

Test ID	Result ID	Reporting Name	Type	Unit	LOINC®
ULFAT	604369	Cryptococcus Ag Titer, LFA, U	Alphanumeric		93766-4

Reference Values:

Negative

Reference values apply to all ages.