

**Test Definition: SHIGC** 

Shigella Culture, Feces

**Reporting Title:** Shigella Culture, F **Performing Location:** Rochester

#### **Additional Testing Requirements:**

If susceptibility testing is needed, also order ZMMLS / Antimicrobial Susceptibility, Aerobic Bacteria, Varies. If susceptibility testing is not needed (eg, due to lack of recovery of *Shigella* species from feces), it will not be performed and the ZMMLS order will be canceled at time of report.

In some cases, local public health requirements may impact Mayo Clinic Laboratories clients, requiring, for example, submission of isolates to public health laboratories. Clients should familiarize themselves with local requirements and are responsible for submitting isolates to appropriate public health laboratories. Clients can obtain isolates of *Shigella* species recovered from specimens submitted to Mayo Clinic Laboratories by calling 800-533-1710 as soon as possible after reporting (to ensure viability of the bacterium).

# **Shipping Instructions:**

Specimen must arrive within 96 hours of collection.

## **Necessary Information:**

Specimen source is required.

## **Specimen Requirements:**

**Patient Preparation:** Medications: Do not use antacids, barium, bismuth, antidiarrheal medication, or oily laxatives before specimen collection.

Supplies: Culture and Sensitivity Stool Transport Vial (T058)

**Container/Tube:** Commercially available transport system specific for recovery of enteric pathogens from fecal specimens (15 mL of non-nutritive transport medium containing phenol red as a pH indicator, either Cary-Blair or Para-Pak C and S)

Specimen Volume: Representative portion of fecal specimen

## **Collection Instructions:**

- 1. Collect fresh feces and submit 1 gram or 5 mL in container with transport medium.
- 2. Place feces in preservative within 2 hours of collection.
- 3. Place vial in a sealed plastic bag.

Specimen Type	Temperature	Time	Special Container
Fecal	Ambient (preferred)	4 days	
	Refrigerated	4 days	

# Ask at Order Entry (AOE) Questions:

Test ID	Question ID	Description	Туре	Reportable
SHIGC	Q00M0085	Specimen Source	Plain Text	Yes

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®



# **Test Definition: SHIGC**

Shigella Culture, Feces

SHIGC Shigella Culture, F	Alphanumeric	mL	88586-3	
---------------------------	--------------	----	---------	--

LOINC® and CPT codes are provided by the performing laboratory.

# **Supplemental Report:**

No

#### **CPT Code Information:**

87046-Shigella Culture, Stool-with isolation and preliminary examination

87077-Bacteria Identification (if appropriate)

87153-Aerobe Ident by Sequencing (if appropriate)

87077-Additional Identification Procedure (if appropriate)

87147 x 4-Serologic Agglut Method 3 Ident (if appropriate)

87077-Ident by MALDI-TOF mass spec (if appropriate)

#### **Reflex Tests:**

Test Id	Reporting Name	CPT Units	CPT Code	Always Performed	Available Separately
GID	Bacteria Identification	1	87077	No	No, (Bill Only)
ISAE	Aerobe Ident by Sequencing	1	87153	No	No, (Bill Only)
REFID	Additional Identification Procedure	1	87077	No	No, (Bill Only)
SHIG	Serologic Agglut Method 3 Ident	1	87147	No	No, (Bill Only)
RMALD	Ident by MALDI-TOF mass spec	1	87077	No	No, (Bill Only)

# **Reference Values:**

No growth of pathogen