

Test Definition: OXU

Oxalate, 24 Hour, Urine

Reporting Title: Oxalate, 24 Hr, U
Performing Location: Rochester

Necessary Information:

24-hour volume (in milliliters) is required.

Specimen Requirements:

Patient Preparation: For 24 hours before, as well as during the collection process, patient should not take large doses (>2 g orally/24 hours) of vitamin C.

Supplies:

- -Diazolidinyl Urea (Germall) 5.0 mL (T822)
- -Sarstedt Aliquot Tube, 5mL (T914)

Container/Tube: Plastic tube or a clean, plastic aliquot container with no metal cap or glued insert

Specimen Volume: 4 mL **Collection Instructions:**

- 1. Add 5 mL of diazolidinyl urea (Germall) as a preservative at start of collection or refrigerate specimen during and after collection.
- 2. Collect urine for 24 hours.
- 3. Mix container thoroughly and aliquot urine into plastic vial.
- 4. Specimen pH should be between 4.5 and 8 and will stay in this range if kept refrigerated. Specimens with pH above 8 indicate bacterial contamination, and testing will be canceled. **Do not** attempt to adjust pH, as it will adversely affect results.

Additional Information: See <u>Urine Preservatives-Collection and Transportation for 24-Hour Urine Specimens</u> for multiple collections.

Forms:

If not ordering electronically, complete, print, and send a Renal Diagnostics Test Request (T830) with the specimen.

Urine Preservative Collection Options:

Note: The addition of preservative **must occur at the start of collection** or application of temperature controls **must occur during and after** collection.

Ambient (No	No
additive)	
Refrigerate (No	OK
additive)	
Frozen (No additive)	No
50% Acetic Acid	No
Boric Acid	No
Diazolidinyl Urea	Preferred
6M Hydrochloric	No
Acid	
6M Nitric Acid	No
Sodium Carbonate	No
Thymol	No
Toluene	No



Test Definition: OXU

Oxalate, 24 Hour, Urine

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	14 days	
	Frozen	14 days	
	Ambient	72 hours	

Ask at Order Entry (AOE) Questions:

Test ID	Question ID	Description	Туре	Reportable
OXU	TM17	Collection Duration	Plain Text	Yes
OXU	VL15	Urine Volume	Plain Text	Yes

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
OCATE	Oxalate, 24 Hr, U (mmol/24 hr)	Numeric	mmol/24 h	14862-7
OXU1	Oxalate, 24 Hr, U (mg/24 hr)	Numeric	mg/24 h	2701-1
TM17	Collection Duration	Numeric	h	13362-9
VL15	Urine Volume	Numeric	mL	3167-4

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

83945

Reference Values:

0.11-0.46 mmol/24 h

9.7-40.5 mg/24 h

The reference value is for a 24-hour collection. Specimens collected for other than a 24-hour period are reported in unit of mmol/L for which reference values are not established.

Reference values have not been established for patients who are younger than 16 years.