

Triglycerides, CDC, Serum

# **Reporting Title:** Triglycerides, CDC, S **Performing Location:** Rochester

## **Specimen Requirements:**

Only orderable as part of a profile. For more information see LMPP / Lipoprotein Metabolism Profile, Serum.

### **Patient Preparation:**

1. Fasting-overnight (12-14 hours)

2. Patient must not consume any alcohol for 24 hours before the specimen is collected.

#### **Collection Container/Tube:**

Preferred: Serum gel

Acceptable: Red top

Submission Container/Tube: Plastic vial

### Specimen Volume: 5 mL

**Collection Instructions:** Centrifuge and aliquot serum into a plastic vial.

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	7 days	
	Frozen	60 days	

## **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
TRIGC	Triglycerides, CDC, S	Numeric	mg/dL	2571-8

LOINC<sup>®</sup> and CPT codes are provided by the performing laboratory.

#### Supplemental Report:

No

## **CPT Code Information:**

84478

## **Reference Values:**

Only orderable as part of a profile. For more information see LMPP / Lipoprotein Metabolism Profile, Serum.

Triglycerides 2-9 years: Acceptable: <75 mg/dL Borderline high: 75-99 mg/dL High: > or =100 mg/dL

10-17 years: Acceptable: <90 mg/dL



# **Test Definition: TRIGC**

Triglycerides, CDC, Serum

Borderline high: 90-129 mg/dL High: > or =130 mg/dL

> or =18 years:
Normal: <150 mg/dL</li>
Borderline high: 150-199 mg/dL
High: 200-499 mg/dL
Very high: > or =500 mg/dL

Reference values have not been established for patients who are less than 2 years of age.