

Reducing Substance, Feces

Reporting Title: Reducing Substance, F **Performing Location:** Rochester

Specimen Requirements:

Supplies: Stool container, Small (Random), 4 oz Random (T288)

Container/Tube: Fecal container

Specimen Volume: 3 g

Collection Instructions:

1. Collect a loose, unpreserved, random fecal specimen.

2. Freeze immediately.

Additional Information: If additional tests are ordered, aliquot and separate sample prior to freezing to allow 1 container per test.

Forms:

If not ordering electronically, complete, print, and send a <u>Gastroenterology and Hepatology Test Request</u> (T728) with the specimen.

Specimen Type	Temperature	Time	Special Container
Fecal	Frozen	7 days	

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
6215	Reducing Substance, F	Alphanumeric		11060-1

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information: 84376

Reference Values:

Negative or trace