

Reporting Title: F2-Isoprostanes, U

Performing Location: Rochester

Specimen Requirements:

**Patient Preparation:** Patient should not take nonsteroidal anti-inflammatory drugs within the 72 hours or aspirin within the 2 weeks prior to specimen collection.

**Supplies:** Sarstedt 5 mL Aliquot Tube (T914)

**Container/Tube:** Plastic urine tube

**Specimen Volume:** 5 mL

Collection Instructions:

- 1. Collect a random urine specimen.
- 2. No preservative.

Forms:

If not ordering electronically, complete, print, and send a [Cardiovascular Test Request Form](#) (T724) with the specimen.

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	7 days	
	Frozen	30 days	
	Ambient	7 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
88677	15-F2t-Isoprostane, U	Alphanumeric	ng/mg Cr	90783-2

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

82542

Reflex Tests:

Test Id	Reporting Name	CPT Units	CPT Code	Always Performed	Available Separately
CRETR	Creatinine, Random, U	1	82570	Yes	No

Result Codes for Reflex Tests:

Test ID	Result ID	Reporting Name	Type	Unit	LOINC®
CRETR	CRETR	Creatinine, Random, U	Numeric	mg/dL	2161-8

Reference Values:

> or =18 years: < or =1.0 ng/mg creatinine  
Reference values have not been established for patients who are younger than 18 years of age