

Reporting Title: Perampanel, S  
Performing Location: Rochester

Specimen Requirements:  
Collection Container/Tube: Red top (serum gel/SST is **not acceptable**)  
Submission Container/Tube: Plastic vial  
Specimen Volume: 1 mL  
Collection Instructions:

- 1. Draw blood immediately before next scheduled dose.
- 2. For sustained-release formulations ONLY, draw blood a minimum of 12 hours after last dose.
- 3. Centrifuge and aliquot serum into a plastic vial within 2 hours of collection.

Forms:  
[If not ordering electronically, complete, print, and send a Therapeutics Test Request](#) (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	28 days	
	Ambient	28 days	
	Frozen	28 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
609438	Perampanel, S	Numeric	ng/mL	88895-8

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:  
No

CPT Code Information:  
80299

Reference Values:  
180-980 ng/mL