

Pancreatic Elastase, Feces

## **Reporting Title:** Pancreatic Elastase, F **Performing Location:** Rochester

### Shipping Instructions:

Preferred shipping temperature is frozen. Refrigerated or ambient specimens received more than 72 hours post collection will be rejected.

#### Specimen Requirements:

Supplies: Stool container, Small (Random), 4 oz (T288)

Container/Tube: Stool container

#### Specimen Volume: 5 g

#### **Collection Instructions:**

- 1. Collect a fresh random fecal specimen, no preservatives.
- 2. If specimen is sent refrigerate, send immediately after collection.
- 3. If specimen cannot be sent immediately, freeze specimen and send frozen.

#### Additional Information:

1. Separate specimens must be submitted when multiple tests are ordered, with the exception of CALPR / Calprotectin,

- Feces. If only a single specimen is collected, it must be split prior to transport.
- 2. Testing cannot be added on to a previously collected specimen.

#### Forms:

If not ordering electronically, complete, print, and send 1 of the following forms with the specimen:

-General Request (T239)

-Gastroenterology and Hepatology Test Request (T728)

Specimen Type	Temperature	Time	Special Container	
Fecal	Frozen (preferred)	28 days		
	Ambient	72 hours		
	Refrigerated	72 hours		

#### Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
ELASF	Pancreatic Elastase, F	Numeric	mcg/g	25907-7

LOINC<sup>®</sup> and CPT codes are provided by the performing laboratory.

#### Supplemental Report:

No

# **CPT Code Information:** 82653

82653

# **Reference Values:**



# **Test Definition: ELASF**

Pancreatic Elastase, Feces

<100 mcg/g (Severe Pancreatic Insufficiency) 100-200 mcg/g (Moderate Pancreatic Insufficiency) >200 mcg/g (Normal)

Reference values apply to all ages.