

Reporting Title: Hepatitis B Past Exposure, S  
Performing Location: Rochester

Necessary Information:  
Date of collection is required.

Specimen Requirements:

Patient Preparation: For 24 hours before specimen collection, patient should **not** take multivitamins or dietary supplements (eg, hair, skin, and nail supplements) containing biotin (vitamin B7).

Supplies: Sarstedt Aliquot Tube, 5 mL (T914)

Collection Container/Tube: Serum gel

Submission Container/Tube: Plastic vial

Specimen Volume: 1.2 mL

Collection Instructions:

- Centrifuge blood collection tube per manufacturer's instructions (eg, centrifuge and aliquot within 2 hours of collection for BD Vacutainer tubes).
- Aliquot serum into plastic vial.

Specimen Type	Temperature	Time	Special Container
Serum SST	Frozen (preferred)	90 days	
	Refrigerated	6 days	
	Ambient	72 hours	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
HBCSN	HBc Total Ab Scrn, S	Alphanumeric		13952-7
HBAGS	HBs Antigen Scrn, S	Alphanumeric		5196-1
HBSQN	HBs Antibody, Quantitative, S	Alphanumeric	mIU/mL	5193-8
HBASN	HBs Antibody Scrn, S	Alphanumeric		10900-9

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:  
No

Components:

Test Id	Reporting Name	CPT Units	CPT Code	Always Performed	Available Separately
HBGSN	HBs Antigen Scrn, S	1	87340	Yes	Yes
HBCSN	HBc Total Ab Scrn, S	1	86704	Yes	Yes
HBBSN	HBs Antibody Scrn, S	1	86706	Yes	Yes

CPT Code Information:

86706  
86704  
87340  
87341 (if appropriate)  
G0499 (if appropriate)

Reflex Tests:

Test Id	Reporting Name	CPT Units	CPT Code	Always Performed	Available Separately
HBGSC	HBs Antigen Screen Confirmation, S	1	87341	No	No

Result Codes for Reflex Tests:

Test ID	Result ID	Reporting Name	Type	Unit	LOINC®
HBGSC	HBGSC	HBs Antigen Screen Confirmation, S	Alphanumeric		7905-3

Reference Values:

Negative  
See [Viral Hepatitis Serologic Profiles](#).