

## **Test Definition: T821Q**

RUNX1-RUNX1T1 Translocation (8;21), Minimal Residual Disease Monitoring, Quantitative, Varies

Reporting Title: RUNX1/RUNX1T1, t(8;21), Quant, V

Performing Location: Rochester

### **Shipping Instructions:**

- 1. Refrigerated specimens must arrive within 5 days of collection, and ambient specimens must arrive within 3 days of collection.
- 2. Collect and package specimen as close to shipping time as possible.

### **Necessary Information:**

### The following information is required:

- 1. Pertinent clinical history
- 2. Date of collection
- 3. Specimen source (blood or bone marrow)

### **Specimen Requirements:**

Submit only 1 of the following specimens:

Specimen Type: Blood

Container/Tube: Lavender top (EDTA) or yellow top (ACD-B)

**Specimen Volume:** 10 mL **Collection Instructions:** 

1. Invert several times to mix blood.

2. Send whole blood specimen in original tube. **Do not aliquot**.

3. Label specimen as blood.

Specimen Type: Bone marrow aspirate

Container/Tube: Lavender top (EDTA) or yellow top (ACD-B)

**Specimen Volume:** 4 mL Collection Instructions:

- 1. Invert several times to mix bone marrow.
- 2. Send bone marrow specimen in original tube. **Do not aliquot**.
- 3. Label specimen as bone marrow.

#### Forms:

- 1. Hematopathology Patient Information (T676)
- 2.<u>If not ordering electronically, complete, print, and send an Hematopathology/Cytogenetics Test Request</u> (T726) with the specimen.

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	5 days	
	Ambient	72 hours	



# **Test Definition: T821Q**

RUNX1-RUNX1T1 Translocation (8;21), Minimal Residual Disease Monitoring, Quantitative, Varies

## Ask at Order Entry (AOE) Questions:

Test ID	Question ID	Description	Туре	Reportable
T821Q	MP061	Specimen Type:	Answer List	Yes
		Peripheral blood		
		Bone marrow		

### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
MP061	Specimen Type	Alphanumeric		31208-2
615906	Interpretation	Alphanumeric		69047-9
616034	Signing Pathologist	Alphanumeric		18771-6

LOINC® and CPT codes are provided by the performing laboratory.

## **Supplemental Report:**

Supplemental

### **CPT Code Information:**

81401

### **Reference Values:**

An interpretive report will be provided.