

# **Test Definition: C1INF**

C1 Esterase Inhibitor, Functional, Serum

Reporting Title: C1 Esterase Inhib, Functional, S

Performing Location: Rochester

**Specimen Requirements:** 

Patient Preparation: Fasting, preferred but not required

Supplies: Sarstedt 5 mL Aliquot Tube (T914)

**Collection Container/Tube:** 

**Preferred:** Red top **Acceptable:** Serum gel

Submission Container/Tube: Plastic vial

**Specimen Volume:** 1 mL **Collection Instructions:** 

1. Immediately after specimen collection, place the tube on wet ice.

2. Centrifuge at 4 degrees C and aliquot serum into a 5-mL plastic vial.

3. Freeze specimen within 30 minutes.

Specimen Type	Temperature	Time	Special Container	
Serum Red	Frozen	28 days		

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
C1INF	C1 Esterase Inhib, Functional, S	Numeric	%	48494-9

LOINC® and CPT codes are provided by the performing laboratory.

### **Supplemental Report:**

Nο

#### **CPT Code Information:**

83520

## **Reference Values:**

>67% normal (normal)

41-67% normal (equivocal)

<41% normal (abnormal)