

**Reporting Title:** Oxycodone Screen, CoC, U  
**Performing Location:** Rochester

**Specimen Requirements:**

**Container/Tube:** Chain-of-Custody Kit (T282) containing the specimen containers, seals, and documentation required.

**Specimen Volume:** 20 mL

**Collection Instructions:** Collect specimen in the container provided, seal, and submit with the associated documentation to satisfy the legal requirements for chain-of-custody testing.

**Forms:**

- 1. [Chain of Custody Request](#) is included in the Chain-of-Custody Kit (T282).
- 2. If not ordering electronically, complete, print, and send a [Therapeutics Test Request](#) (T831) with the specimen.

| Specimen Type | Temperature              | Time     | Special Container |
|---------------|--------------------------|----------|-------------------|
| Urine         | Refrigerated (preferred) | 14 days  |                   |
|               | Frozen                   | 14 days  |                   |
|               | Ambient                  | 72 hours |                   |

**Result Codes:**

| Result ID | Reporting Name   | Type         | Unit  | LOINC®  |
|-----------|------------------|--------------|-------|---------|
| 61727     | Oxycodone        | Alphanumeric | ng/mL | 19642-8 |
| 36027     | Chain of Custody | Alphanumeric |       | 77202-0 |

LOINC® and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

**CPT Code Information:**

80307

**Reflex Tests:**

| Test Id | Reporting Name                      | CPT Units | CPT Code | Always Performed | Available Separately |
|---------|-------------------------------------|-----------|----------|------------------|----------------------|
| OXYCX   | Oxycodone w/metabolite Conf, CoC, U | 1         | 80365    | No               | Yes                  |

**Result Codes for Reflex Tests:**

| Test ID | Result ID | Reporting Name   | Type    | Unit  | LOINC® |
|---------|-----------|------------------|---------|-------|--------|
| ADLTX   | 36121     | Creatinine, U    | Numeric | mg/dL | 2161-8 |
| ADLTX   | 36122     | Specific Gravity | Numeric |       | 5810-7 |

|       |       |                              |              |       |         |
|-------|-------|------------------------------|--------------|-------|---------|
| ADLTX | 36123 | pH                           | Numeric      |       | 2756-5  |
| ADLTX | 36124 | Oxidants                     | Alphanumeric |       | 58714-7 |
| ADLTX | 36125 | Nitrites                     | Alphanumeric |       | 32710-6 |
| ADLTX | 36126 | Comment                      | Alphanumeric |       | 48767-8 |
| OXYCX | 36300 | Oxycodone Immunoassay Screen | Alphanumeric | ng/mL | 19642-8 |
| OXYCX | 61728 | Oxycodone-by LC-MS/MS        | Alphanumeric | ng/mL | 16249-5 |
| OXYCX | 35964 | Oxymorphone-by LC-MS/MS      | Alphanumeric | ng/mL | 17395-5 |
| OXYCX | 36023 | Oxycodone Interpretation     | Alphanumeric |       | 69050-3 |
| OXYCX | 36028 | Chain of Custody             | Alphanumeric |       | 77202-0 |
| OXYCX | 42012 | Noroxycodone-by LC-MS/MS     | Alphanumeric | ng/mL | 61425-5 |
| OXYCX | 42013 | Noroxymorphone-by LC-MS/MS   | Alphanumeric | ng/mL | 90894-7 |

Reference Values:

Negative  
Screening cutoff concentration:  
Oxycodone: 100 ng/mL