

Reporting Title: Oxycodone w/metabolite Conf, CoC, U
Performing Location: Rochester

Specimen Requirements:

Supplies: Chain-of-Custody Kit (T282)
Container/Tube: Chain-of-custody kit containing the specimen containers, seals, and documentation required.
Specimen Volume: 5 mL
Collection Instructions: Collect urine specimen in the container provided, seal, and submit with the associated documentation to satisfy the legal requirements for chain-of-custody testing.

Forms:

- 1. [Chain of Custody Request](#) is included in the Chain-of-Custody Kit (T282).
- 2. If not ordering electronically, complete, print, and send a [Therapeutics Test Request](#) (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	28 days	
	Frozen	28 days	
	Ambient	14 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
36300	Oxycodone Immunoassay Screen	Alphanumeric	ng/mL	19642-8
61728	Oxycodone-by LC-MS/MS	Alphanumeric	ng/mL	16249-5
35964	Oxymorphone-by LC-MS/MS	Alphanumeric	ng/mL	17395-5
36023	Oxycodone Interpretation	Alphanumeric		69050-3
36028	Chain of Custody	Alphanumeric		77202-0
42012	Noroxycodone-by LC-MS/MS	Alphanumeric	ng/mL	61425-5
42013	Noroxymorphone-by LC-MS/MS	Alphanumeric	ng/mL	90894-7

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

80365
G0480 (if appropriate)

Reflex Tests:

Test Id	Reporting Name	CPT Units	CPT Code	Always Performed	Available Separately
---------	----------------	-----------	----------	------------------	----------------------

Test Definition: OXYCX

Oxycodone with Metabolite Confirmation,
Chain of Custody, Random, Urine

COCH	Chain of Custody Processing	1	99001	Yes	No
ADLTX	Adulterants Survey, CoC, U	1	81005	Yes	Yes

Result Codes for Reflex Tests:

Test ID	Result ID	Reporting Name	Type	Unit	LOINC®
ADLTX	36121	Creatinine, U	Numeric	mg/dL	2161-8
ADLTX	36122	Specific Gravity	Numeric		5810-7
ADLTX	36123	pH	Numeric		2756-5
ADLTX	36124	Oxidants	Alphanumeric		58714-7
ADLTX	36125	Nitrites	Alphanumeric		32710-6
ADLTX	36126	Comment	Alphanumeric		48767-8

Reference Values:

Negative

Cutoff concentrations:

Oxycodone Immunoassay screen: 100 ng/mL

By liquid chromatography-tandem mass spectroscopy:

Oxycodone: 25 ng/mL

Noroxycodone: 25 ng/mL

Oxymorphone: 25 ng/mL

Noroxymorphone: 25 ng/mL