

Test Definition: PEPAN

Comprehensive Peripheral Neuropathy Gene Panel, Varies

Reporting Title: Peripheral Neuropathy Gene Panel **Performing Location:** Rochester

Ordering Guidance:

Targeted testing for familial variants (also called site-specific or known mutations testing) is available for the genes on this panel. See FMTT / Familial Variant, Targeted Testing, Varies. To obtain more information about this testing option, call 800-533-1710.

Customization of this panel and single gene analysis for any gene present on this panel are available. For more information see CGPH/ Custom Gene Panel, Hereditary, Next-Generation Sequencing, Varies.

Shipping Instructions:

Specimen preferred to arrive within 96 hours of collection.

Specimen Requirements:

Patient Preparation: A previous bone marrow transplant from an allogenic donor will interfere with testing. Call 800-533-1710 for instructions for testing patients who have received a bone marrow transplant.

Specimen Type: Whole blood

Container/Tube: Lavender top (EDTA) or yellow top (ACD)

Acceptable: Any anticoagulant

Specimen Volume: 3 mL

Collection Instructions:

1. Invert several times to mix blood.

2. Send whole blood specimen in original tube. Do not aliquot.

Specimen Stability Information: Ambient (preferred)/Refrigerated

Forms:

1. New York Clients-Informed consent is required. Document on the request form or electronic order that a copy is on file.

The following documents are available:

-Informed Consent for Genetic Testing (T576)

-Informed Consent for Genetic Testing (Spanish) (T826)

2. Molecular Genetics: Neurology Patient Information

3. If not ordering electronically, complete, print, and send a <u>Neurology Specialty Testing Client Test Request</u> (T732) with the specimen.

Specimen Type	Temperature	Time	Special Container
Varies	Varies		

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
617689	Test Description	Alphanumeric		62364-5
617690	Specimen	Alphanumeric		31208-2

 \mathcal{P} MAYO CLINIC LABORATORIES

Test Definition: PEPAN

Comprehensive Peripheral Neuropathy Gene Panel, Varies

617691	Source	Alphanumeric	31208-2
617692	Result Summary	Alphanumeric	50397-9
617693	Result	Alphanumeric	82939-0
617694	Interpretation	Alphanumeric	69047-9
618189	Additional Results	Alphanumeric	82939-0
617695	Resources	Alphanumeric	99622-3
617696	Additional Information	Alphanumeric	48767-8
617697	Method	Alphanumeric	85069-3
617698	Genes Analyzed	Alphanumeric	48018-6
617699	Disclaimer	Alphanumeric	62364-5
617700	Released By	Alphanumeric	18771-6

LOINC[®] and CPT codes are provided by the performing laboratory.

Supplemental Report:

Supplemental

CPT Code Information: 81448

Reference Values:

An interpretive report will be provided.