

Reporting Title: Protein, Total, Random, U
Performing Location: Rochester

Specimen Requirements:
Only orderable as part of a profile. For more information see:
ORTHP / Orthostatic Proteinuria, Random, Urine

Patient Preparation: Specimens should be collected before fluorescein is given or not collected until at least 24 hour later.

Container/Tube: Plastic, 5-mL tube

Specimen Volume: 4 mL

Collection Instructions:

- 1. This is a nighttime (supine) collection.
- 2. Void immediately prior to lying down.
- 3. Collect an 8-hour (nighttime) urine specimen.
- 4. Upon awaking in the morning, the patient should immediately void into the specimen cup.
- 4. No preservative.
- 5. Invert well before taking 4-mL (maximum) aliquot.

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	14 days	
	Frozen	30 days	
	Ambient	24 hours	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
PRCON	Protein, Total, Random, U	Numeric	mg/dL	2888-6

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:
No

CPT Code Information:
84156

Reference Values:
Not established