

Reporting Title: NGPCM Pre-Analysis Cell Sorting, BM
Performing Location: Rochester

Specimen Requirements:
Only orderable as a reflex. For more information see NGPCM / MayoComplete Plasma Cell Myeloma, Next-Generation Sequencing, Varies.

Specimen Type: Bone marrow aspirate
Container/Tube: Lavender or pink top (EDTA) or yellow top (ACD)
Specimen Volume: 2 mL
Collection Instructions:

- 1. Minimum plasma cell percentage is 5%.
- 2. Invert several times to mix bone marrow.
- 3. Send bone marrow specimen in original tube. **Do not aliquot.**
- 4. Label specimen as bone marrow.
- 5. **Fresh specimen is required for this test**, as testing is performed on sorted cells.

Specimen Stability Information: Ambient (preferred) 4 days/Refrigerate

Specimen Type	Temperature	Time	Special Container
Bone Marrow	Ambient	4 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
618627	NGPCM Pre-Analysis Cell Sort	Alphanumeric		No LOINC Needed
618630	Final Diagnosis	Alphanumeric		22637-3

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:
No

CPT Code Information:
88184-Flow Cytometry; first cell surface, cytoplasmic or nuclear marker
88185 x 5-Flow Cytometry, additional cell surface, cytoplasmic or nuclear marker (each)

Reference Values:
Only orderable as a reflex. For more information see NGPCM / MayoComplete Plasma Cell Myeloma, Next-Generation Sequencing, Varies.

Not applicable