

Cystic Fibrosis Transmembrane Conductance Regulator, CFTR, Full Gene Analysis, Varies

Reporting Title: CFTR Gene, Full Gene Analysis

Performing Location: Rochester

Ordering Guidance:

This test is not the preferred test for cystic fibrous carrier screening. See CFMP / Cystic Fibrosis, *CFTR* Gene, Variant Panel, Varies.

Targeted testing for familial variants (also called site-specific or known variants testing) is available for variants identified in the CFTR gene. See FMTT / Familial Variant, Targeted Testing, Varies. To obtain more information about testing option, call 800-533-1710.

Additional Testing Requirements:

All prenatal specimens must be accompanied by a maternal blood specimen; order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen. **This must be a different order number than the prenatal specimen**.

For cord blood specimens: Maternal cell contamination (MCC) studies are available. **Order MATCC on both the cord blood and maternal specimens under separate order numbers**. Cord blood testing will proceed without MCC studies, but results may be compromised if MCC is present.

Shipping Instructions:

Specimen preferred to arrive within 96 hours of collection.

Specimen Requirements:

Patient Preparation: A previous bone marrow transplant from an allogenic donor will interfere with testing. For instructions for testing patients who have received a bone marrow transplant, call 800-533-1710.

Submit only 1 of the following specimens:

Specimen Type: Whole blood

Container/Tube:

Preferred: Lavender top (EDTA) or yellow top (ACD)

Acceptable: Any anticoagulant Specimen Volume: 3 mL Collection Instructions:

1. Invert several times to mix blood.

2. Send whole blood specimen in original tube. Do not aliquot.

Specimen Stability Information: Ambient (preferred) 4 days/Refrigerated

Specimen Type: Skin biopsy

Supplies: Fibroblast Biopsy Transport Media (T115)

Container/Tube: Sterile container with any standard cell culture media (eg, minimal essential media, RPMI 1640). The

solution should be supplemented with 1% penicillin and streptomycin.

Specimen Volume: 4-mm punch

Specimen Stability Information: Refrigerated (preferred)/Ambient



Cystic Fibrosis Transmembrane Conductance Regulator, CFTR, Full Gene Analysis, Varies

Additional Information: A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks is required to culture fibroblasts before genetic testing can occur.

Specimen Type: Cultured fibroblasts

Container/Tube: T-25 flask Specimen Volume: 2 Flasks

Collection Instructions: Submit confluent cultured fibroblast cells from a skin biopsy from another laboratory.

Specimen Stability Information: Ambient (preferred)/Refrigerated (<24 hours)

Additional Information: A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or

Molecular Testing. An additional 3 to 4 weeks is required to culture fibroblasts before genetic testing can occur.

Due to its complexity, consultation with the laboratory is required for all prenatal testing; call 800-533-1710 to speak to a genetic counselor.

Specimen Type: Amniotic fluid

Container/Tube: Amniotic fluid container

Specimen Volume: 20 mL

Specimen Stability Information: Refrigerated (preferred)/Ambient

Additional information:

- 1. A separate culture charge will be assessed under CULAF / Culture for Genetic Testing, Amniotic Fluid. An additional 2 to 3 weeks is required to culture amniotic fluid before genetic testing can occur.
- **2. All prenatal specimens must be accompanied by a maternal blood specimen;** order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Specimen Type: Chorionic villi

Container/Tube: 15-mL tube containing 15 mL of transport media

Specimen Volume: 20 mg

Specimen Stability Information: Refrigerated

Additional Information:

- 1. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 2 to 3 weeks is required to culture chorionic villi before genetic testing can occur.
- **2. All prenatal specimens must be accompanied by a maternal blood specimen;** order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Specimen Type: Confluent cultured cells

Container/Tube: T-25 flask Specimen Volume: 2 Flasks

Collection Instructions: Submit confluent cultured cells from another laboratory.

Specimen Stability Information: Ambient (preferred)/Refrigerated

Additional Information: All prenatal specimens must be accompanied by a maternal blood specimen; order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Forms:

1. New York Clients-Informed consent is required. Document on the request form or electronic order that a copy is on



Cystic Fibrosis Transmembrane Conductance Regulator, CFTR, Full Gene Analysis, Varies

file. The following documents are available:

- -Informed Consent for Genetic Testing (T576)
- -Informed Consent for Genetic Testing (Spanish) (T826)
- 2. Molecular Genetics: Congenital Inherited Diseases Patient Information (T521)

Specimen Type	Temperature	Time	Special Container
Varies	Varies		

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
619775	Test Description	Alphanumeric		62364-5
619776	Specimen	Alphanumeric		31208-2
619777	Source	Alphanumeric		31208-2
619778	Result Summary	Alphanumeric		50397-9
619779	Result	Alphanumeric		82939-0
619780	Interpretation	Alphanumeric		69047-9
619781	Additional Results	Alphanumeric		82939-0
619782	Resources	Alphanumeric		99622-3
619783	Additional Information	Alphanumeric		48767-8
619784	Method	Alphanumeric		85069-3
619785	Genes Analyzed	Alphanumeric		82939-0
619786	Disclaimer	Alphanumeric		62364-5
619787	Released By	Alphanumeric		18771-6

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

Supplemental

CPT Code Information:

81223

88233- Tissue culture, skin, solid tissue biopsy (if appropriate)

88240- Cryopreservation (if appropriate)

Reflex Tests:

Test Id	Reporting Name	CPT Units	CPT Code	Always Performed	Available Separately
_STR1	Comp Analysis using STR (Bill only)	1	81265	No	No, (Bill only)
_STR2	Add'l comp analysis w/STR (Bill Only)	1	81266	No	No, (Bill only)
CULFB	Fibroblast Culture for Genetic Test	1	88233	No	Yes



Cystic Fibrosis Transmembrane Conductance Regulator, CFTR, Full Gene Analysis, Varies

CULAF	Amniotic Fluid Culture/Genetic Test	1	88235	No	Yes
MATCC	Maternal Cell Contamination, B	1	81265	No	Yes

Result Codes for Reflex Tests:

Test ID	Result ID	Reporting Name	Type Unit		LOINC®
CULAF	52304	Result Summary	Alphanumeric		50397-9
CULAF	52306	Interpretation	Alphanumeric	Alphanumeric	
CULAF	52305	Result	Alphanumeric	Alphanumeric	
CULAF	CG767	Reason for Referral	Alphanumeric		42349-1
CULAF	52307	Specimen	Alphanumeric		31208-2
CULAF	52308	Source	Alphanumeric		31208-2
CULAF	52309	Method	Alphanumeric		85069-3
CULAF	54641	Additional Information	Alphanumeric		48767-8
CULAF	52310	Released By	Alphanumeric		18771-6
CULFB	52327	Result Summary	Alphanumeric		50397-9
CULFB	52329	Interpretation	Alphanumeric		69965-2
CULFB	52328	Result	Alphanumeric		82939-0
CULFB	CG770	Reason for Referral	Alphanumeric		42349-1
CULFB	CG899	Specimen	Alphanumeric		31208-2
CULFB	52331	Source	Alphanumeric		31208-2
CULFB	52332	Method	Alphanumeric		85069-3
CULFB	54625	Additional Information	Alphanumeric		48767-8
CULFB	52333	Released By	Alphanumeric		18771-6
MATCC	53285	Result Summary	Alphanumeric		50397-9
MATCC	53286	Result	Alphanumeric		40704-9
MATCC	53287	Interpretation	Alphanumeric		69047-9
MATCC	53288	Reason for referral	Alphanumeric		42349-1
MATCC	53289	Specimen	Alphanumeric		31208-2
MATCC	53290	Source	Alphanumeric		31208-2
MATCC	53291	Released By	Alphanumeric		18771-6
MATCC	55150	Method	Alphanumeric		85069-3

Reference Values:

An interpretive report will be provided