

**Test Definition: FERR1** 

Ferritin, Serum

**Reporting Title:** Ferritin, S **Performing Location:** Rochester

## **Specimen Requirements:**

**Patient Preparation: For 12 hours before specimen collection do not** take multivitamins or dietary supplements containing biotin (vitamin B7), which is commonly found in hair, skin, and nail supplements and multivitamins.

**Collection Container/Tube:** 

**Preferred:** Serum gel **Acceptable:** Red top

**Submission Container Tube:** Plastic vial

Specimen Volume: 0.6 mL

Collection Instructions: Centrifuge and aliquot serum into a plastic vial within 2 hours of collection.

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	7 days	
	Frozen	365 days	
	Ambient	24 hours	

### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
FERR1	Ferritin, S	Numeric	mcg/L	20567-4

LOINC® and CPT codes are provided by the performing laboratory.

# **Supplemental Report:**

No

### **CPT Code Information:**

82728

### **Reference Values:**

Males:

0 days-4 weeks: 150-973 mcg/L 5 weeks-5 months: 9-580 mcg/L 6 months-9 years: 6-111 mcg/L 10-17 years: 15-201 mcg/L > or =18 years: 31-409 mcg/L

Females:

0 days-4 weeks: 150-973 mcg/L 5 weeks-5 months: 9-580 mcg/L 6 months-17 years: 8-115 mcg/L

18-50 years: 6-175 mcg/L



**Test Definition: FERR1** 

Ferritin, Serum

> or =51 years: 11-328 mcg/L