

Test Definition: DSSX

Drug Screen, Prescription/Over the Counter, Chain of Custody, Serum

Reporting Title: Drug Scrn, Prescription/OTC, CoC, S **Performing Location:** Rochester

Specimen Requirements:

Supplies: Chain-of-Custody Kit (T282)

Container/Tube: Chain-of-Custody Kit containing the specimen containers, seals, and documentation required.

Preferred: 10-mL Red top (serum gel/SST are not acceptable)

Acceptable: 5-mL Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 2.75 mL

Collection Instructions:

1. Centrifuge and aliquot serum into plastic vial within 2 hours of collection.

2. Cap, seal, and submit with the associated documentation to satisfy the legal requirements for chain-of-custody testing.

Additional Information: See Emergency/Overdose.

Forms:

<u>1. Chain of Custody Request</u> is included in the Chain-of-Custody Kit (T282).

2. If not ordering electronically, complete, print, and send a <u>Therapeutics Test Request</u> (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	14 days	
	Frozen	14 days	
	Ambient	3 hours	

Ask at Order Entry (AOE) Questions:

Test ID	Question ID	Description	Туре	Reportable
DSSX	SUDR	Suspect Drug	Plain Text	Yes

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
36185	Drugs detected:	Alphanumeric		20785-2
36186	Chain of Custody	Alphanumeric		77202-0

LOINC[®] and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:



Test Definition: DSSX

Drug Screen, Prescription/Over the Counter, Chain of Custody, Serum

80307

Reflex Tests:

Test Id	Reporting Name	CPT Units	CPT Code	Always Performed	Available Separately
COCH	Chain of Custody Processing	1	99001	Yes	No

Reference Values:

Drugs detected are presumptive. Additional testing may be required to confirm the presence of any drugs detected.