

Test Definition: MAMMX

6-Monoacetylmorphine (6-MAM) Confirmation, Chain of Custody, Meconium

Reporting Title: 6-MAM Confirmation, CoC, M

Performing Location: Rochester

Shipping Instructions:

The laboratory recommends sending chain-of-custody specimens by overnight shipment.

Specimen Requirements:

Supplies: Chain-of-Custody Meconium Kit (T653) includes the specimen containers, seals, and documentation required.

Specimen Volume: 1 g (approximately 1 teaspoon)

Collection Instructions:

- 1. Collect entire random meconium specimen.
- 2. Send specimen frozen. When refrigerated, a significant percentage of 6-monoacetylmorphine will convert to morphine in less than 24 hours.

Additional Information: Specimen that arrives with a broken seal does not meet the chain of custody requirements.

Forms.

- 1. Chain of Custody Request is included in the Chain-of-Custody Kit (T282).
- 2. If not ordering electronically, complete, print, and send a <u>Therapeutics Test Request</u> (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Meconium	Frozen	14 days	

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
36202	6-Monoacetylmorphine	Alphanumeric	ng/g	29345-6
36203	Interpretation	Alphanumeric		69050-3
36204	Chain of Custody	Alphanumeric		77202-0

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

80356

G0480 (if appropriate)

Reflex Tests:

Test Id	Reporting Name	CPT Units	CPT Code	Always Performed	Available Separately
COCH	Chain of Custody Processing	1	99001	Yes	No



Test Definition: MAMMX

6-Monoacetylmorphine (6-MAM) Confirmation, Chain of Custody, Meconium

R	efe	ren	ıce	٧a	alu	es:

Negative

Positives are reported with a quantitative liquid chromatography tandem mass spectrometry result. Cutoff concentration: 5 ng/g