

Test Definition: ETGS

Ethyl Glucuronide Screen, Random, Urine

Reporting Title: Ethyl Glucuronide Screen, U **Performing Location:** Rochester

Ordering Guidance:

For situations where chain of custody is required, a Chain of Custody Kit (T282) is available. For chain-of-custody testing, order ETGX / Ethyl Glucuronide Confirmation, Chain of Custody, Random, Urine.

Additional Testing Requirements:

Specimen Requirements:

Supplies: Sarstedt 5 mL Aliquot Tube (T914) Collection Container/Tube: Plastic urine container Submission Container/Tube: Plastic, 5-mL tube Specimen Volume: 2 mL

Collection Instructions:

- 1. Collect a random urine specimen.
- 2. No preservative.

Forms:

If not ordering electronically, complete, print, and send a <u>Therapeutics Test Request</u> (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container	
Urine	Refrigerated (preferred)	28 days		
	Frozen	28 days		
	Ambient	72 hours		

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
63420	Ethyl Glucuronide Screen, U	Alphanumeric	ng/ml	58375-7

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information: 80307

80307

Reference Values:

Negative

Screening cutoff concentration: 500 ng/mL