

**Reporting Title:** Adalimumab QN with Reflex to Ab, S  
**Performing Location:** Rochester

**Ordering Guidance:**  
If both quantitation and antibody testing are needed, regardless of the quantitation results, order ADALP / Adalimumab Quantitative with Antibody, Serum

**Specimen Requirements:**  
**Patient Preparation:** For 12 hours before specimen collection, it is recommended that the patient not take multivitamins or dietary supplements containing biotin (vitamin B7), which is commonly found in hair, skin, and nail supplements and multivitamins.  
**Collection Container/Tube:**  
**Preferred:** Serum gel  
**Acceptable:** Red top  
**Submission Container/Tube:** Plastic vial  
**Specimen Volume:** 0.5 mL  
**Collection Instructions:** Centrifuge and aliquot serum into a plastic vial

**Forms:**  
If not ordering electronically, complete, print, and send one of the following with the specimen:  
[-Gastroenterology and Hepatology Test Request](#) (T728)  
[-Therapeutics Test Request](#) (T831)

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Frozen	28 days	

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
ADALX	Adalimumab QN with Reflex to Ab, S	Numeric	mcg/mL	86894-3

LOINC® and CPT codes are provided by the performing laboratory.

**Supplemental Report:**  
No

**CPT Code Information:**  
80145  
83520 (if appropriate)

**Reflex Tests:**

Test Definition: ADALX

Adalimumab Quantitative with Reflex to  
Antibody, Serum

Test Id	Reporting Name	CPT Units	CPT Code	Always Performed	Available Separately
ADLAB	Adalimumab Ab, S	1	83520	No	No

Result Codes for Reflex Tests:

Test ID	Result ID	Reporting Name	Type	Unit	LOINC®
ADLAB	ADLAB	Adalimumab Ab, S	Numeric	AU/mL	90779-0

Reference Values:

ADALIMUMAB QUANTITATIVE:

Limit of quantitation is 0.8 mcg/mL.Optimal therapeutic ranges are disease specific.

ADALIMUMAB ANTIBODY:

<14.0 AU/mL