

## **Test Definition: MPNCM**

Myeloproliferative Neoplasm, CALR with Reflex to MPL, Varies

Reporting Title: MPN (CALR, MPL) Reflex

Performing Location: Rochester

### **Shipping Instructions:**

Specimen must arrive within 7 days of collection.

#### **Necessary Information:**

#### The following information is required:

- 1. Pertinent clinical history
- 2. Clinical or morphologic suspicion
- 3. Date of collection
- 4. Specimen source

## **Specimen Requirements:**

Submit only 1 of the following specimens:

Specimen Type: Blood

Container/Tube: Lavender top (EDTA) or yellow top (ACD-B)

**Specimen Volume:** 3 mL **Collection Instructions:** 

1. Invert several times to mix blood.

- 2. Send specimen in original tube.
- 3. Label specimen as blood.

Specimen Stability Information: Ambient (preferred)/Refrigerate

**Specimen Type:** Bone marrow aspirate

**Container/Tube:** Lavender top (EDTA) or yellow top (ACD-B)

**Specimen Volume:** 2 mL Collection Instructions:

- 1. Invert several times to mix specimen.
- 2. Send specimen in original tube.
- 3. Label specimen as bone marrow.

Specimen Stability Information: Ambient (preferred)/Refrigerate

Specimen Type: Extracted DNA from blood or bone marrow

Container/Tube: 1.5 to 2 mL tube Specimen Volume: Entire specimen

**Collection Instructions:** 

- 1. Indicate volume and concentration of DNA
- 2. Label specimen as extracted DNA from blood or bone marrow.

Specimen Stability Information: Frozen (preferred)/Refrigerate/Ambient

#### Forms:

If not ordering electronically, complete, print, and send a <u>Hematopathology/Cytogenetics Test Request</u> (T726) with the specimen.



# **Test Definition: MPNCM**

Myeloproliferative Neoplasm, CALR with Reflex to MPL, Varies

| Specimen Type | Temperature | Time   | Special Container |
|---------------|-------------|--------|-------------------|
| Varies        | Varies      | 7 days |                   |

## Ask at Order Entry (AOE) Questions:

| Test ID | Question ID | Description   | Туре       | Reportable |
|---------|-------------|---------------|------------|------------|
| MPNCM   | MP036       | Specimen Type | Plain Text | Yes        |

#### **Result Codes:**

| Result ID | Reporting Name      | Туре         | Unit | LOINC®  |
|-----------|---------------------|--------------|------|---------|
| 42393     | MPNCM Reflex Result | Alphanumeric |      | 82939-0 |
| MP036     | Specimen Type       | Alphanumeric |      | 31208-2 |
| 42392     | Final Diagnosis     | Alphanumeric |      | 50398-7 |

LOINC® and CPT codes are provided by the performing laboratory.

## **Supplemental Report:**

Nο

## **CPT Code Information:**

81219-CALR (calreticulin) (eg, myeloproliferative disorders), gene analysis, common variants in exon 9 81339 -MPL (MPL proto-oncogene, thrombopoietin receptor) (eg, myeloproliferative disorder) gene analysis; sequence analysis, exon 10 (if appropriate)

#### **Reflex Tests:**

| Test Id | Reporting Name                 | CPT Units | CPT Code | Always Performed | Available Separately |
|---------|--------------------------------|-----------|----------|------------------|----------------------|
| MPNML   | MPL Exon 10 Sequencing, Reflex | 1         | 81339    | No               | No, (bill only)      |

## **Reference Values:**

An interpretive report will be provided.