

# **Test Definition: CORTF**

Cortisol, Free, Serum

**Reporting Title:** Cortisol, Free, S **Performing Location:** Rochester

#### **Additional Testing Requirements:**

## **Necessary Information:**

Include time of collection.

**Specimen Requirements:** 

**Collection Container/Tube:** Red top **Submission Container/Tube:** Plastic vial

**Specimen Volume:** 1.25 mL **Collection Instructions:** 

1. Morning (8 a.m.) specimens are preferred. The 8 a.m. cortisol can be referred to as the a.m. cortisol and can be collected any time between 6 a.m. and 10:30 a.m. in the morning.

2. Centrifuge and aliquot serum into a plastic vial.

**Additional Information**: If multiple specimens are collected, send a separate order for each specimen.

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	28 days	
	Ambient	28 days	
	Frozen	28 days	

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
65423	Cortisol, Free, S	Numeric	mcg/dL	2145-1

LOINC® and CPT codes are provided by the performing laboratory.

### **Supplemental Report:**

No

#### **CPT Code Information:**

82530

## **Reference Values:**

6-10:30 a.m. Collection: 0.121-1.065 mcg/dL