

Test Definition: TRPS

Troponin T, 5th Generation, Plasma

Reporting Title: Troponin T, 5th gen, P

Performing Location: Rochester

Specimen Requirements: Collection Container/Tube:

Preferred: Light-green top (lithium heparin gel)

Acceptable: Green top (lithium heparin)
Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL **Collection Instructions:**

- 1. Lithium heparin gel tubes should be centrifuged within 2 hours of collection.
- 2. Plasma from lithium heparin tubes should be centrifuged and aliquoted into a plastic vial within 2 hours of collection.

Forms:

If not ordering electronically, complete, print, and send a <u>Cardiovascular Test Request Form</u> (T724) with the specimen.

Specimen Type	Temperature	Time	Special Container
Plasma Li Heparin	Frozen (preferred)	365 days	
	Ambient	24 hours	
	Refrigerated	24 hours	

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
TRPS	Troponin T, 5th gen, P	Numeric	ng/L	67151-1

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

84484

Reference Values:

Males: < or =15 ng/L Females: < or =10 ng/L