

**Reporting Title:** IGG Subtypes Immunofluorescence  
**Performing Location:** Rochester

**Ordering Guidance:**

If additional interpretation/analysis is needed, request PATHC / Pathology Consultation along with this test and send the corresponding renal pathology light microscopy and immunofluorescence (IF) slides (or IF images on a CD), electron microscopy images (prints or CD), and the pathology report.

**Shipping Instructions:**

- 1. Advise shipping specimens in Styrofoam transportation coolers to avoid extreme hot or cold temperatures to ensure specimens are received at required specimen stability temperature.
- 2. Attach the green pathology address label included in the kit to the outside of the transport container.

**Necessary Information:**

A pathology/diagnostic report is required.

**Specimen Requirements:**

**Preferred:** Frozen tissue

**Supplies:** Renal Biopsy Kit (T231)

**Specimen Type:** Kidney tissue

**Container/Tube:** Renal Biopsy Kit, Zeus/Michel's, Frozen

**Specimen Volume:** Entire specimen

**Collection Instructions:** Collect specimens according to the instructions in [Renal Biopsy Procedure for Handling Tissue for Light Microscopy, Immunofluorescent Histology, and Electron Microscopy](#).

**Additional Information:** If standard immunoglobulin and complement immunofluorescence has already been performed, submit the residual frozen tissue (must contain glomeruli) on dry ice.

**Acceptable:** Frozen tissue

**Slides:** 4 frozen tissue unstained positively-charged glass slides (25- x 75- x 1-mm) per test ordered; sections 4-microns thick.

**Forms:**

- 1. [Renal Biopsy Patient Information](#)
- 2. If not ordering electronically, complete, print, and send a [Renal Diagnostics Test Request](#) (T830) with the specimen.

Specimen Type	Temperature	Time	Special Container
Special	Frozen (preferred)		
	Ambient		
	Refrigerated		

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
BA0271	Interpretation	Alphanumeric		50595-8

BA0272	Participated in the Interpretation	Alphanumeric		No LOINC Needed
BA0276	Addendum	Alphanumeric		35265-8
BA0275	Report electronically signed by	Alphanumeric		19139-5
71618	Disclaimer	Alphanumeric		62364-5
71850	Case Number	Alphanumeric		80398-1
BA0274	Gross Description	Alphanumeric		22634-0
BA0273	Material Received	Alphanumeric		22633-2

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

88346-primary IF

88350-if additional IF