

# **Test Definition: FCARO**

Carotene, Beta

Reporting Title: Carotene, Beta
Performing Location: LabCorp Burlington

**Specimen Requirements:** 

Supplies: Sarstedt 5 mL Aliquot Tube (Amber) (T915)

Specimen Type: Serum Container/Tube: Red Specimen Volume:1 mL

Collection Instructions: Draw blood in a plain, red-top tube(s). (Serum gel tube is acceptable.)

Spin Down and send 1 mL of serum ambient in an amber vial to protect from light.

#### Note:

- 1. Protect from light within 1 hour of collection.
- 2. Patient must be fasting overnight (12 hours). Blood should be collected before breakfast in the morning and prior to any medication.
- 3. Abstain from alcohol for 24 hours prior to collection.

Specimen Type	Temperature	Time	Special Container	
Serum	Ambient (preferred)	14 days	LIGHT PROTECTED	
	Frozen	14 days	LIGHT PROTECTED	
	Refrigerated	14 days	LIGHT PROTECTED	

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
FCARO	Carotene, Beta	Numeric	ug/dL	2053-7

LOINC® and CPT codes are provided by the performing laboratory.

## **Supplemental Report:**

No

### **CPT Code Information:**

82380

## **Reference Values:**

3 - 91 ug/dL