

Reporting Title: Prostaglandin D2 (PG D2), U
Performing Location: Inter Science Institute

Specimen Requirements:

Patient Preparation: Patient should not be on aspirin, indomethacin, or anti-inflammatory medications, if possible, for at least 48 hours prior to collection of specimen. Specimen must be frozen within 30 minutes of collection.

Specimen Type: Urine

Submission Container/Tube: Plastic, 10-mL tube (T068)

Specimen Volume: 10 mL

Collection Instructions:

- 1. Collect random urine (NO preservative).
- 3. Freeze immediately and send specimen frozen in the plastic, 10-mL urine tube (T068)

Note: 24 hours urine collection is not acceptable.

Specimen Type	Temperature	Time	Special Container
Urine	Frozen	180 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
FPD2U	Prostaglandin D2 (PG D2), U	Alphanumeric	ng/g Creatinine	Not Provided

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

84150

Reference Values:

Up to 175 ng/g Creatinine