

# **Test Definition: FDGTX**

Digitoxin, Serum

Reporting Title: Digitoxin
Performing Location: NMS Labs

### **Specimen Requirements:**

\*\*\*Must submit one specimen per order. Specimens cannot be shared between multiple orders.\*\*\*

**Specimen Type:** Serum

Collection Container/Tube: Red top Submission Container/Tube: Plastic vial

**Specimen Volume:** 1 mL **Collection Instructions:** 

- 1. Draw blood in a plain, red-top tube(s). Serum gel tube is not acceptable.
- 2. Centrifuge and send 1 mL of serum refrigerated in a plastic, preservative-free vial.

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	7 days	
	Frozen	90 days	

### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
FDGTX	Digitoxin	Alphanumeric		3559-2

LOINC® and CPT codes are provided by the performing laboratory.

## **Supplemental Report:**

Referral

### **CPT Code Information:**

80299

#### **Reference Values:**

Reporting limit determined each analysis.

None Detected ng/mL

Therapeutic Range: 10 - 30 ng/mL