

# **Test Definition: FD2PG**

Prostaglandin D2 (PGD2)

Reporting Title: Prostaglandin D2 (PGD2)
Performing Location: Inter Science Institute

# **Specimen Requirements:**

# **Patient preparation:**

Patient should NOT be on any aspirin, indomethacin, or anti-inflammatory medications, if possible, for at least 48 hours prior to collection of specimen.

Specimen Type: Serum Container/Tube: Red top Specimen Volume: 3 mL

Collection Instructions: Draw blood in a plain, red-top tube(s), serum-gel tube(s) is acceptable. Separate immediately

and send 3 mL of serum frozen in a plastic vial.

Specimen Type	Temperature	Time	Special Container	
Serum	Frozen	60 days		

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
FD2PG	Prostaglandin D2 (PGD2)	Alphanumeric	pg/mL	12839-7

LOINC® and CPT codes are provided by the performing laboratory.

# **Supplemental Report:**

No

# **CPT Code Information:**

84150

# **Reference Values:**

Adult Reference Range(s): 35 - 115 pg/mL

No pediatric reference ranges available for this test.