

Reporting Title: Warfarin, Plasma
Performing Location: NMS Labs

Specimen Requirements:

****Must submit one specimen per order. Specimens cannot be shared between multiple orders.****

Collection Container/Tube: EDTA (lavender top) or pink top

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Collection Instructions:

- 1. Draw blood in EDTA (lavender top) or pink top tube(s). **Plasma gel tube is not acceptable.**
- 2. Centrifuge and send 1 mL of plasma refrigerate in preservative-free plastic vial.

Specimen Type	Temperature	Time	Special Container
Plasma EDTA	Refrigerated (preferred)	30 days	
	Ambient	30 days	
	Frozen		

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
Z5526	Warfarin	Alphanumeric		4098-0

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

80375

Reference Values:

Reporting limit determined each analysis.
None Detected mcg/mL