

Cannabidiol, Serum

**Reporting Title:** Cannabidiol, Serum **Performing Location:** NMS Labs

# **Specimen Requirements:**

\*\*\*Must submit one specimen per order. Specimens cannot be shared between multiple orders.\*\*\*

Specimen Type: Serum

Collection Container/Tube: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

## **Collection Instructions:**

- 1. Draw blood in a red-top tube(s). Serum gel tube is not acceptable.
- 2. Centrifuge and send 1 mL of serum refrigerated in a plastic, preservative-free vial.

### Forms:

If not ordering electronically, complete, print, and send a <u>Therapeutics Test Request</u> (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	30 days	
	Ambient	14 days	
	Frozen		

## **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
Z5547	Cannabidiol	Alphanumeric		82964-8

LOINC® and CPT codes are provided by the performing laboratory.

### Supplemental Report:

No

CPT Code Information:

80349 G0480, if appropriate

### **Reference Values:**

Reporting limit determined each analysis Units: ng/mL