

Reporting Title: Anti-TIF-1gamma Ab
Performing Location: Esoterix Endocrinology

Specimen Requirements:

Collection Container/Tube: Red
Submission Container/Tube: Plastic vial
Specimen Volume: 1 mL
Acceptable: SST

Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube is acceptable. Spin down and send 1 mL of serum refrigerated in a plastic vial.

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	14 days	
	Frozen	60 days	
	Ambient	7 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
FAT1G	Anti-TIF-1gamma Ab	Alphanumeric		96501-2

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

86235

Reference Values:

Reference Range: <20

Interpretation:

- Negative: <20 units
- Weak Positive: 20 - 39 units
- Moderate Positive: 40 - 80 units
- Strong Positive: >80 units