

Reporting Title: Pancreastatin, Plasma
Performing Location: Inter Science Institute

Specimen Requirements:

Patient Preparation:

- 1. Patient should not be on any medications that may influence Insulin levels, if possible, for at least 48 hours prior to collection.
- 2. Patient should be fasting 10 hours prior to collection. Patient may drink plain water; no other liquid is acceptable.

Specimen Type: Plasma

Collection Container/Tube: Z tube (MCL T701)

Submission Container/Tube: Plastic vial

Specimen Volume: 3 mL

Collection Instructions:

- 1. Draw 10 mL of blood in special Z-tube, pre-chilled (MCL T701).
- 2. Separate plasma from cells immediately after draw and freeze immediately after separation.
- 3. Send 3 mL of plasma frozen in a plastic vial.

Specimen Type	Temperature	Time	Special Container
Z-Tube Plasma	Frozen	60 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
FPANP	Pancreastatin, Plasma	Numeric	pg/mL	49013-6

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

83519

Reference Values:

10-135 pg/mL