

Reporting Title: Azole Susceptibility Panel

Performing Location: U.T. Health Science Ctr San Antonio

- Shipping Instructions:
- 1. See [Infectious Specimen Shipping Guidelines](#) in Special Instructions.
 - 2. Place specimen in a large infectious container (T146) and label as an etiologic agent/infectious substance.

- Specimen Requirements:
- Container/Tube: Agar slant or other appropriate media
- Specimen volume: Organism in pure culture
- Complete and submit with specimen:
- 1.Fungus Testing Lab request form

- Collection Instructions:
- 1. Organism must be in pure culture, actively growing.
 - 2. Place specimen in a large infectious container (T146) and label as etiologic agent/infectious substance.

Specimen Type	Temperature	Time	Special Container
Varies	Ambient		

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
FAZSP	Azole Susceptibility Panel	Alphanumeric		Not Provided

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

Referral

- CPT Code Information:
- 87999-Unlisted Microbiology Procedure (Refer to patient report to apply the appropriate CPT code in place of this unlisted CPT code based on specimen organism.)
- 87188 x 4 for mould MIC microdilution or agar dilution (if appropriate)
- 87186 x 4 for yeast MIC microdilution or agar dilution (if appropriate)