

Reporting Title: Trimethoprim, S

Performing Location: Rochester

Specimen Requirements:

Collection Container/Tube: Red top (gel tubes/SST are **not** acceptable)

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Collection Instructions:

- Serum for a peak level should be collected at least 60 minutes after a dose.
- Centrifuge and aliquot serum into plastic vial within 2 hours of collection.

Forms:

If not ordering electronically, complete, print, and send a [Therapeutics Test Request](#) (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	28 days	
	Ambient	28 days	
	Frozen	28 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
80146	Trimethoprim, S	Numeric	mcg/mL	11005-6

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

80299

Reference Values:

>2.0 mcg/mL (Peak)