

Reporting Title: Fluoxetine, S
Performing Location: Rochester

Specimen Requirements:
Collection Container/Tube: Red top (serum gel/SST are **not acceptable**)
Submission Container/Tube: Plastic vial
Specimen Volume: 1 mL
Collection Instructions:
1. Draw blood immediately before the next scheduled dose (trough).
2. Centrifuge and aliquot serum into plastic vial within 2 hours of collection.

Forms:
If not ordering electronically, complete, print, and send a [Therapeutics Test Request](#) (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	28 days	
	Frozen	28 days	
	Ambient	72 hours	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
80228	Fluoxetine, S	Numeric	ng/mL	74982-0
251	Norfluoxetine, S	Numeric	ng/mL	3868-7
252	Fluoxetine+Norfluoxetine	Alphanumeric	ng/mL	74948-1

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:
No

CPT Code Information:
80299

Reference Values:
Fluoxetine + Norfluoxetine: 120-500 ng/mL