

Reporting Title: 17-Hydroxypregnenolone, S

Performing Location: Rochester

Specimen Requirements:

Collection Container/Tube:

Preferred: Red top

Acceptable: Serum gel

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Collection Instructions: Centrifuge and aliquot serum into plastic vial.

Specimen Type	Temperature	Time	Special Container
Serum	Frozen (preferred)	28 days	
	Refrigerated	28 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
81151	17-Hydroxypregnenolone, S	Numeric	ng/dL	6765-2

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

84143

Reference Values:

CHILDREN*

Males

Premature (26-28 weeks): 1,219-9,799 ng/dL

Premature (29-36 weeks): 346-8,911 ng/dL

Full term (1-5 months): 229-3,104 ng/dL

6 months-364 days: 221-1,981 ng/dL

1-2 years: 35-712 ng/dL

3-6 years: <277 ng/dL

7-9 years: <188 ng/dL

10-12 years: <393 ng/dL

13-15 years: 35-465 ng/dL

16-17 years: 32-478 ng/dL

TANNER STAGES

Stage I: <209 ng/dL

Stage II: <356 ng/dL
Stage III: <451 ng/dL
Stage IV-V: 35-478 ng/dL

Females

Premature (26-28 weeks): 1,219-9,799 ng/dL
Premature (29-36 weeks): 346-8,911 ng/dL
Full term (1-5 months): 229-3,104 ng/dL
6 months-364 days: 221-1,981 ng/dL
1-2 years: 35-712 ng/dL
3-6 years: <277 ng/dL
7-9 years: <213 ng/dL
10-12 years: <399 ng/dL
13-15 years: <408 ng/dL
16-17 years: <424 ng/dL

TANNER STAGES

Stage I: <236 ng/dL
Stage II: <368 ng/dL
Stage III: <431 ng/dL
Stage IV-V: <413 ng/dL

ADULTS**Males**

> or =18 years: 55-455 ng/dL

Females

> or =18 years: 31-455 ng/dL

*Kushnir MM, Rockwood AL, Roberts WL, et al: Development and performance evaluation of a tandem mass spectrometry assay for 4 adrenal steroids. Clin Chem. 2006 Aug;52(8):1559-1567

To convert to nmol/L, multiply the value in ng/dL by 0.03159757.