

Test Definition: LAGU

Legionella Antigen, Random, Urine

Reporting Title: Legionella Ag, U **Performing Location:** Rochester

Ordering Guidance:

This assay has been verified using urine specimens only. For serum specimens, order SLEG / Legionella pneumophila (Legionnaires Disease), Antibody, Serum.

Other specimen types (eg, plasma or body fluids) that may contain Legionella antigen have not been verified for testing.

Specimen Requirements:

Supplies: Urine Tubes, 10 mL (T068) **Container/Tube:** Plastic, 10-mL urine tube

Specimen Volume: 0.5 mL **Collection Instructions:**

- 1. Collect a random urine specimen.
- 2. No preservative.
- 3. Excessively bloody or very turbid specimens containing protein, cells, or particulates will be canceled. They can inhibit the function of the test.
- 4. Centrifuging to remove particulates is not approved.
- 5. Specimens with any dyes or unnatural color are not acceptable and will be canceled.

Forms:

If not ordering electronically, complete, print, and send 1 of the following with the specimen:

- -Infectious Disease Serology Test Request (T916)
- -Microbiology Test Request (T732)

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	7 days	
	Frozen	14 days	
	Ambient	24 hours	

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
81268	Legionella Ag, U	Alphanumeric		32781-7

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

87899



Test Definition: LAGU

Legionella Antigen, Random, Urine

Reference Values	R	efer	ence	Val	lues	•
------------------	---	------	------	-----	------	---

Negative