

Reporting Title: Complement, Total, S

Performing Location: Rochester

Specimen Requirements:

Patient Preparation: Fasting preferred.

Supplies: Sarstedt 5 mL Aliquot Tube (T914)

Collection Container/Tube: Red top (serum gel/SST are **not** acceptable)

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Collection Instructions:

1. Immediately after specimen collection, place the tube on wet ice.
2. Centrifuge and aliquot serum into plastic vial.
3. Immediately freeze specimen.

Forms:

Specimen Type	Temperature	Time	Special Container
Serum Red	Frozen	28 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
COM	Complement, Total, S	Numeric	U/mL	4532-8

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

86162

Reference Values:

30-75 U/mL