

Reporting Title: Supplemental Newborn Screen, BS
Performing Location: Rochester

Additional Testing Requirements:

A repeat specimen is required within 1 week of birth for infants tested before they are 12 hours old.

Specimen Requirements:

Patient must be older than 12 hours and less than 1 week of age.

Supplies: Card-Blood Spot Collection Filter Paper (T493)

Preferred: Blood Spot Collection Card

Acceptable: Whatman Protein Saver 903 Paper, Munktell, PerkinElmer 226 filter paper, or local newborn screening card

Specimen Volume: 3 Blood spots

Collection Instructions:

1. Do not use device or capillary tube containing EDTA to collect specimen.
2. Completely fill at least 3 circles on the filter paper card (approximately 100 microliters blood per circle).
3. Let blood dry on the Blood Spot Collection Card at ambient temperature in a horizontal position for 3 hours.
4. Do not expose specimen to heat or direct sunlight.
5. Do not stack wet specimens.
6. Keep specimen dry.

Additional Information:

1. For collection instructions, see [Blood Spot Collection Instructions](#).
2. For collection instructions in Spanish, see [Blood Spot Collection Card-Spanish Instructions](#) (T777).
3. For collection instructions in Chinese, see [Blood Spot Collection Card-Chinese Instructions](#) (T800).

Forms:

[If not ordering electronically, complete, print, and send a Biochemical Genetics Test Request](#) (T798) with the specimen.

Specimen Type	Temperature	Time	Special Container
Whole blood	Ambient (preferred)		FILTER PAPER
	Frozen		FILTER PAPER
	Refrigerated		FILTER PAPER

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
82594	Supplemental Newborn Screen Result	Alphanumeric		54089-8
23727	Reviewed By	Alphanumeric		18771-6

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

83789

Reference Values:

An interpretive report will be provided.